



Unit 5, Alma Park Road, Alma Park Industrial Estate, Grantham, Lincolnshire, NG31 9SE  
Reception: 01476 564 921 Sales: 01476 564 963 Fax: 01476 564 302

E-mail: [info@wmscallsystems.com](mailto:info@wmscallsystems.com)

## PM2 Superior Pressure Mat



*The pressure mat simply plugs into your existing call point and activates a standard call when pressure is applied to the mat. Supplied with a 3m lead terminated with a moulded plug and measuring 93cm x 64cm the large surface area of the pressure mat ensures sensor activation and that the alarm will be raised.*

**Set up and test:** *Place the pressure mat onto a flat surface in the required position. Plug the lead into the jack socket on the front of the Nurse Call unit, step onto the mat and the red reassurance LED on the call point should flash. Step off the mat and press the reset button on the call point to cancel the alarm. This simple test must be carried out prior to every use of the mat. If at any time this mat does not function properly during testing, replace with a new mat and contact your supplier.*

**Safety Information:** *Do not bend the mat as this will cause internal damage and may prevent the unit from operating correctly. Do not place the mat on an uneven surface and always ensure the lead is routed carefully across the room to avoid hazards.*