


Services Nurse Call, Surgery Call, Warden Call, Staff Safety, Access Control, CCTV

Intercall 600	✖
Intercall 700	✓

L762 Audio Call/Display Unit.

The L762 Call/Display Unit provides all the combined features of the L752 Audio call point and L758 Audio LCD display. It features seven levels of call, hands free two way intercom, integral infra red receiver, integral LCD display, audio page facility, audible call re-assurance, re-assurance LED and 'another call' sounder. The trigger socket will accept pear leads, pressure mats, portable radio triggers and a host of other triggering devices. The unit flush mounts into a standard double gang backbox or may be surface mounted using the Intercall BB1. All X1 to X5 remote trigger inputs feature a secondary description facility where details for the remote device may be programmed. For example "En-Suite" or "Ceiling Pull Switch" may be allocated to these inputs. Please refer to the information contained on page 22 for more information regarding installation and performance of the Infra Red receiver.

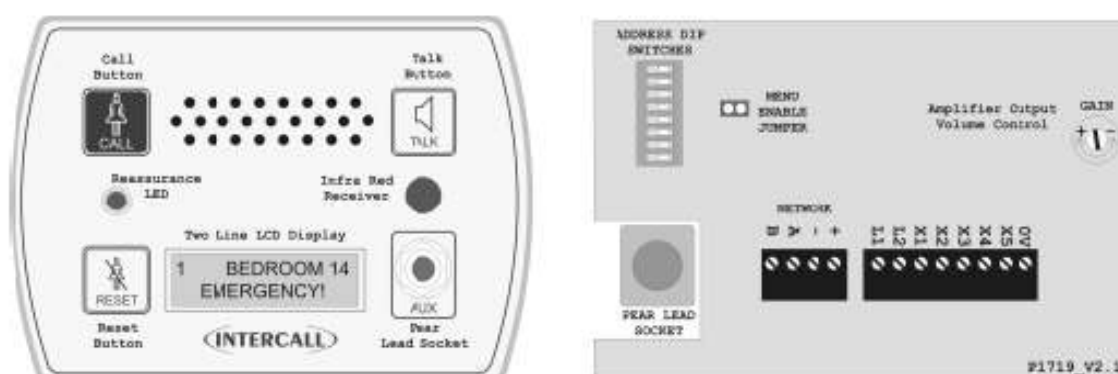

Network Terminals: Connect to the Intercall 700 4 wire Network. [A,B,+, -] OBSERVE POLARITY

X1 Terminal: Programmable Input No 1 – Factory Default is Standard Call, System Text 1
X2 Terminal: Programmable Input No 2 – Factory Default is Emergency Call, System Text 1
X3 Terminal: Programmable Input No 3 – Factory Default is Nurse Present/Reset Toggle.
X4 Terminal: Programmable Input No 4 – Factory Default is Emergency Call, System Text 2
X5 Terminal: Programmable Input No 5 – Factory Default is Nurse Present Only.

L1 Terminal: Red LED Output for Ceiling pull switch LED. Max load 12v 10mA
L2 Terminal: Green LED Output for Slave Reset Point LED. Max load 12v 10mA
0V Terminal: Additional Network negative common terminal for auxiliary connections.

Address Switches: Set to unique setting between 1 and 215.

Re-assurance LED: Slow Red flashing – Standard Call
 Faster Red flashing – Priority Call
 Slow Red/Green flashing – Assistance Call
 Rapid Red flashing – Emergency/Crash Call
 Green flashing - Accepted Call – Staff on their way / Paging from Call Point
 Constant Green – Nurse Present Mode

Audio gain control: Controls the speech level output from the call point. This does not effect the volume from the speaker in the call point.

IR Receiver: Typically, the unit will cover a room 6m x 6m. **Important: See Page 22.**

Integral Menus: Extensive menu structure for Call Point & Display configuration. See Page 57

Installation: Flush mount using standard 'double size' electrical backbox or surface mount using BB1

Size & Weights: 160 x 115 x 25 mm 135g